GB

AIRMAX NASAL DILATOR

The Airmax generates a scientifically proven increase in airflow of 76.1% on average by widening the nasal passage at the narrowest point of the nose. In this way, it helps reduce snoring and improve sleep quality and increases the intake of air during exercise. The product has been developed by airflow experts in collaboration with ENT specialists. The Airmax is made of soft, odourless and medically certified plastic (S.E.B.S.), making it extremely comfortable and responsible to wear.

INSTRUCTIONS OF USE

Rinse the Airmax with water before and after use. Do not use harsh cleaning agents or disinfectants on it. Use a mirror to correctly position the Airmax and check the illustration for reference. Use the Airmax for at least two consecutive weeks to get used to it. The Airmax may fall out at night; this will decrease over time as use increases. Note: The Sport version is fitted with small 'barbs' to prevent it from falling out of your nose during exercise.

RECOMMENDED DURATION OF USE OF 2 MONTHS
Preferably store the product in a dry and dark place to preserve its duration of use. We recommend replacing the Airmax every 2 months for optimal hygiene, comfort and effectiveness.

CAUTIONS

The Airmax is only suitable for positioning in the nose and not suitable for children under the age of 12 years. Keep the Airmax out of the reach of small children. The Airmax should be used by one person only to avoid contamination. If you experience any difficulties using the Airmax, we recommend that you consult an ENT specialist. No allergic reactions or other contraindications are known so far. Any serious incident in connection with this device should be reported to the manufacturer and the competent authorities of the Member State where the user is located. When disposing of this device, the general legal requirements for waste.

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