# DermaForte® Wrat weg

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www.dermaforte.com

# DermaForte® Wart away

## For the **treatment** of common **warts** on hands and feet

### **Dab only once** **Painless and effective** **For children and adults**

**General advice:** Carefully read the instructions for use prior to using DermaForte® Wart away, even if you used this medical device before. The information may have been amended according to new research and findings in the time since you opened your last package. This medical device can be obtained without a prescription. In spite of this, take care when using DermaForte® Wart away in order to obtain a good result. Keep these instructions for use in case you need to consult them again.

**Application:** DermaForte® Wart away is a medical device for the treatment of common warts. It contains monochloroacetic acid, a substance having a degrading action on the skin. The wart will come loose, layer after layer. Monochloroacetic acid will make sure that the cells containing the wart virus disappear.

**The benefits:** One benefit of using DermaForte® Wart away is that dabbing the wart does not hurt, which means that it can also be used for children.

- Suitable for children from the age of 4
- Only 1 treatment per week
- Painless dabbing
- Effective for thick warts on palms of hands and soles of feet as well as stubborn warts
- Easy and simple to use
- Child safe packaging

**Instructions for use:** DermaForte® Wart away has a child safe cap that can be opened as follows:

Place the bottle on a solid underground that is able to resist the corrosive fluid. Avoid contact with clothes, textile or table-cloths. You can open the bottle by pressing down the cap and turning it simultaneously. Remove the cap from the bottle as follows:

Use DermaForte® Wart away only once per week. In order to prevent the skin surrounding the wart from being affected, we recommend protecting the skin surrounding the wart against the corrosive action of the fluid, prior to the treatment, for instance by means of Vaseline or zinc ointment. First, thoroughly pat dry the spatula, e.g. by means of paper towels, and then return the spatula in the bottle so that a little bit of fluid attaches to it. In this way you will not apply too much fluid to the wart. Then, apply a small amount of DermaForte® Wart away to the wart using the spatula (do not allow the fluid to drip). Make sure that you only apply DermaForte® Wart away to the wart. Allow it to dry for a moment. There is no need to cover the wart. After a week, the wart will come loose, layer after layer. The fluid will need a week to soak off the top layer of the skin, so you will not see an immediate result after application. After some time the wart cells are destroyed and the wart will get a white colour and will be rejected. DermaForte® Wart away should not be applied more than once per week. Depending on the size and thickness of the wart, it may be necessary to repeat the treatment with DermaForte® Wart away after one week. Warts are contagious. That is why we do not recommend using the product for more than one person.

Please apply DermaForte® Wart away with utmost care. The solution is corrosive. Be extra careful when applying it to body parts that are more difficult to reach, such as soles of feet. This particularly applies to persons with a limited mobility or in case of disturbances in the motor system (trembling hands, stiff fingers). Carefully close the bottle after use.

**Ingredients:** monochloroacetic acid, purified water.

### **Do not use DermaForte® Wart away:**

- If you are oversensitive to one or more of the ingredients;
- In the face (the fluid should never make contact with the eyes or the mucous membrane in the nose or mouth);

- For genital warts or in the genital area;
- On birthmarks, open wounds, moles, naevus, old age (brown) warts, darkly coloured, hairy or strange looking warts;
- On red, irritated or infected warts;
- Immediately after a surgical removal of warts, freezing of warts (cryotherapy) or a laser treatment. This particularly applies when the skin is red or painful after the treatment. In this case you will have to wait for at least 4 weeks, until the redness or inflammation of the skin has fully disappeared. Only then you can start using DermaForte® Wart away;
- During pregnancy or while breastfeeding.

### **Be extra careful when using DermaForte® Wart away:**

- In case of doubts concerning the diagnosis (the change in the skin cannot be clearly explained as a wart);
- In case of immune deficiency (e.g. when you are using cytostatic medication);
- In case of diabetes, or other complaints such as vascular diseases, as a result of which wounds will heal less easily.

In these cases, DermaForte® Wart away may not be used before a doctor has been consulted.

### **Warnings:**

- Only intended for external use;
- Use with utmost care. The fluid of DermaForte® Wart away is corrosive;
- If the healthy skin makes contact with DermaForte® Wart away, please rinse it with water. If, in spite of these precautions, the skin becomes irritated, it will have to undergo the same treatment as burns;
- Contact your general practitioner, if your complaints continue to exist;
- Prevent skin contact with the bottle's screw thread;
- Properly close the bottle after use and store it in an upright position.

**Use in combination with other drugs:** As far as we know, there are no interactions with drugs.

**Possible side effects:** DermaForte® Wart away may cause some side effects, although not everyone will experience these side effects. The side effects that could occur when using DermaForte® Wart away are: irritated skin, burning sensation, skin redness or oversensitivity reactions. If any of these side effects becomes serious, or if you notice any side effects not listed in these instructions for use, please consult your doctor or pharmacist.

**Content:** 4 ml

**Storage instructions:** Store at a temperature of 15-25°C in a dry and dark place in the original container and out of reach of young children. Do not use after the expiry date stated on the packaging.

**Medical device:** Produced in accordance with the strict quality standards of ISO 13485: 2003. DermaForte® Wart away is a class I medical device in compliance with the medical device directive 93/42/EC, amended by MDD 2007/47/EEC.

**The DermaForte® information line:** For specific questions about this medical device you can call the information line: 0900-5826848. This special information number can be reached during office hours.

**Final date of use:** We advise against using DermaForte® Wart away after the expiry date, which is stated on the packaging after 'expiry date'. DermaForte® Wart away can be kept for 1 month after opening.

Date on which these instructions for use were last revised: May 2012

IFU: DF05-12

Peters Krizman AG  
Gessnerallee 38, CH-8001 Zürich, CH

**Distr. Netherlands:** PK Benelux BV  
Vluchtoord 17, 5406 XP Uden, NL

www.dermaforte.com